



Compressed Gauze Users Manual

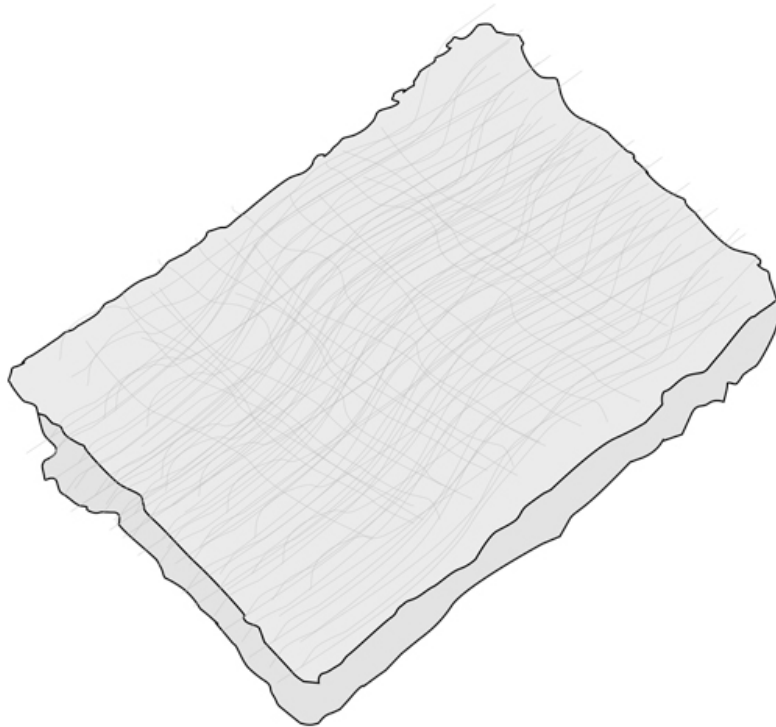


Compressed, rolled gauze is ideal for wound bandaging and wrapping of injuries.


Unique crinkle weave ensures excellent fluid absorption.


Sterile & Latex Free.

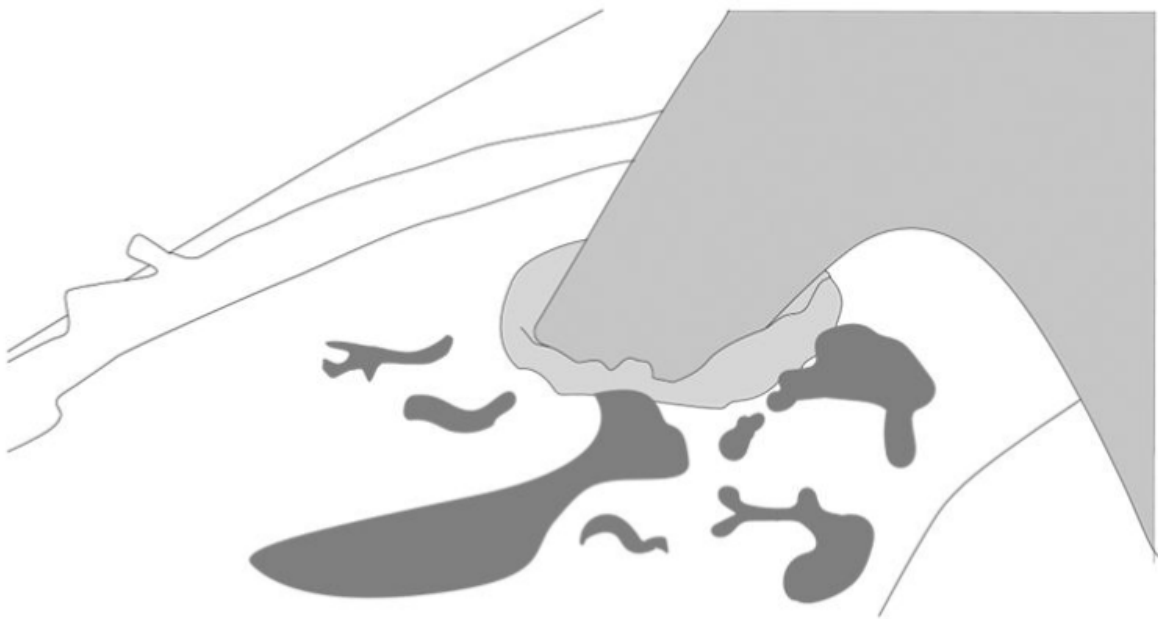

□ Caption Original

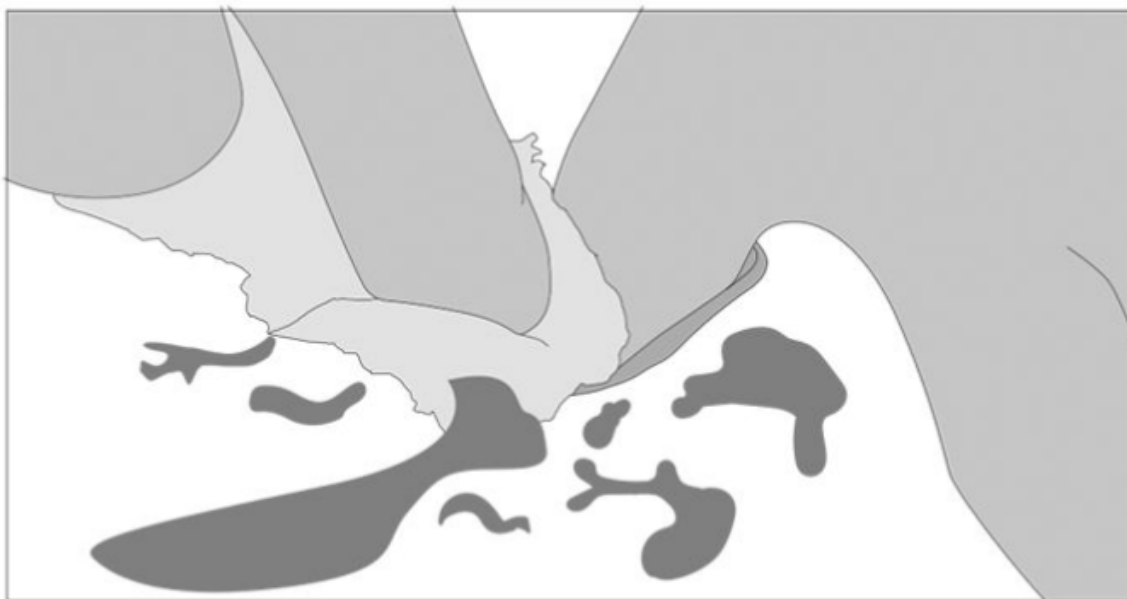

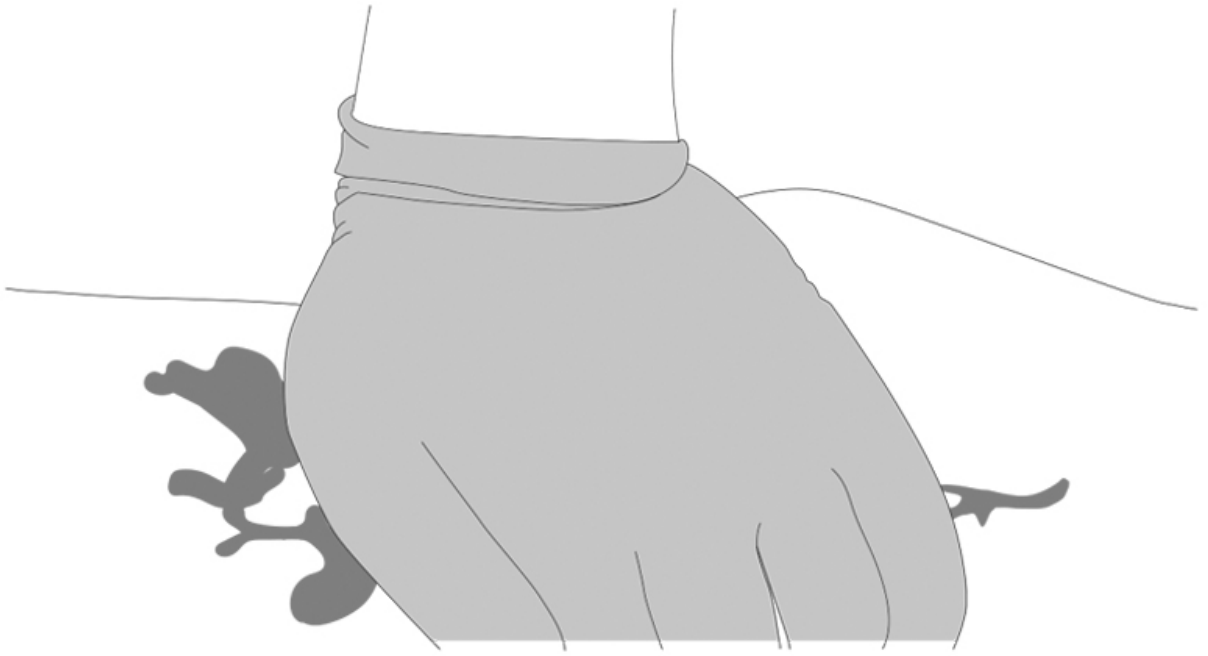

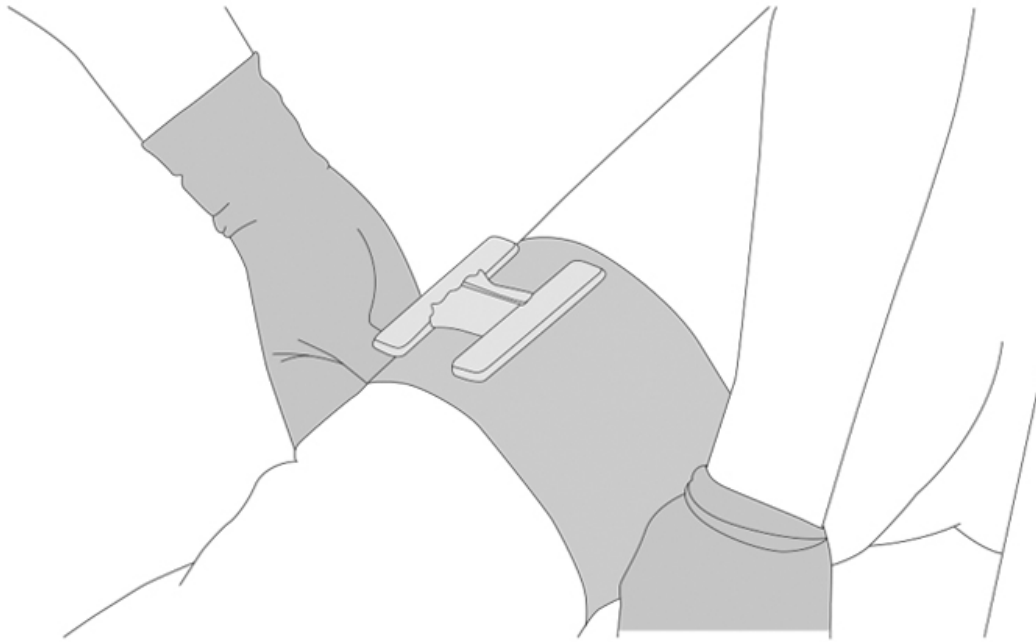

Direction to Use Compressed Gauze

Step 1: Pull 6 to 8 inches of packing material out from the initial pack

Step 2: Probe the wound gently to check for shrapnel, find the source of bleeding and apply pressure

Step 3: Feed the pack into the wound until you reach the end of the wound

Step 4: When the wound is completely packed, apply pressure for ten minutes if no compression bandage is available

Step 5: Apply a trauma bandage to keep the gauze in place