


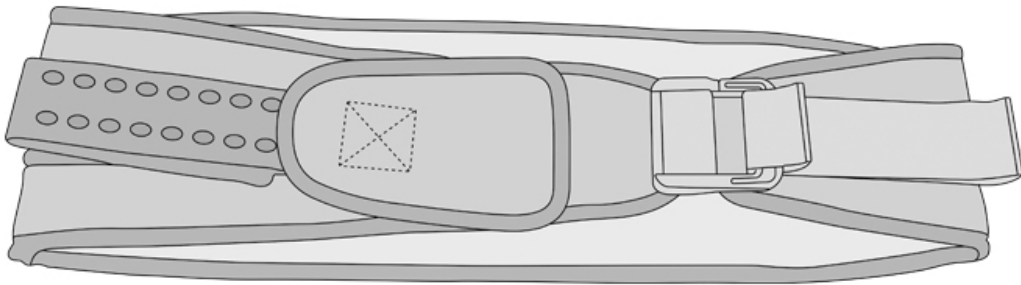


Pelvic Sling Users Manual

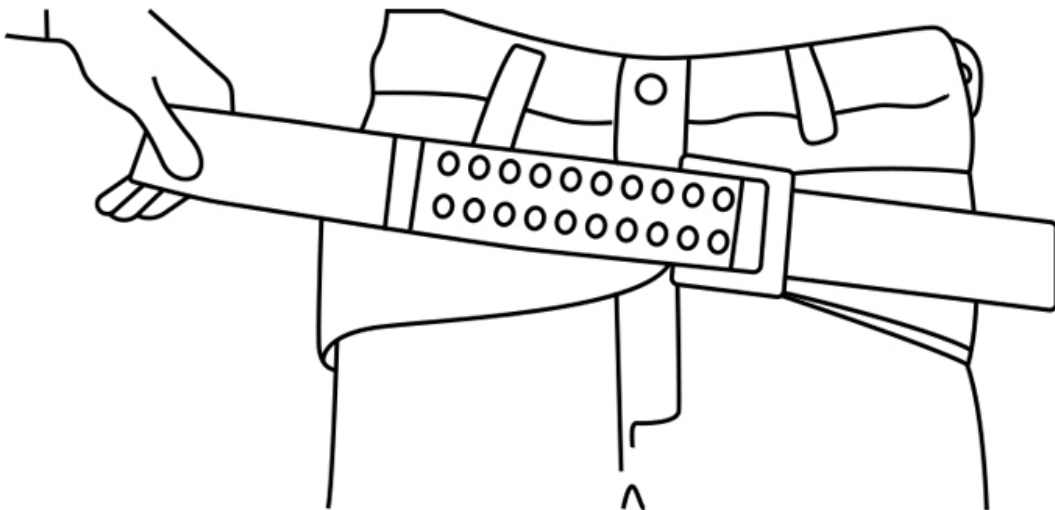

A pelvic sling provides support to keep the urethra closed when sneezing and coughing. It has a low risk of complications and a high rate of effectiveness.

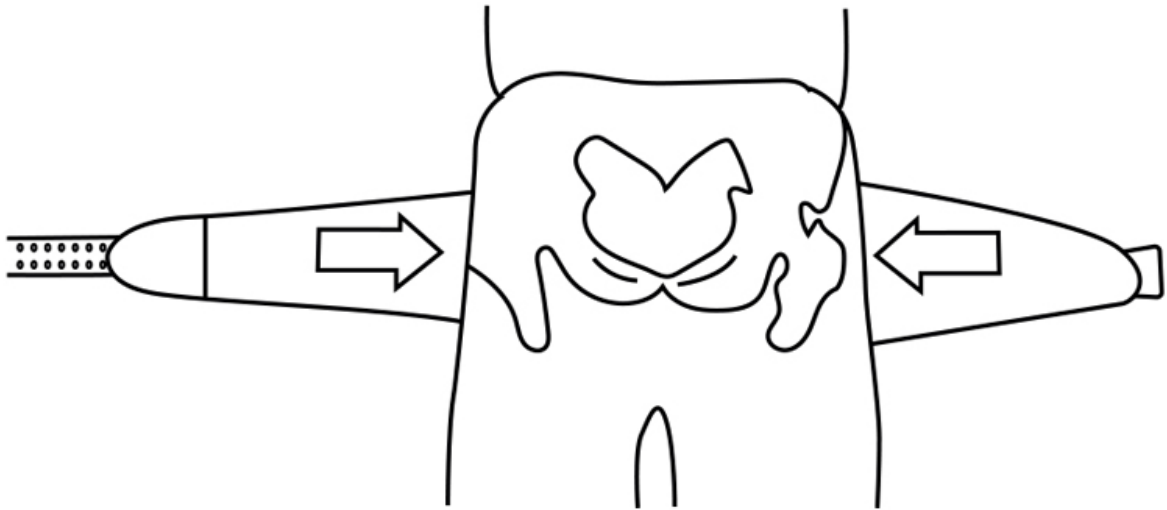

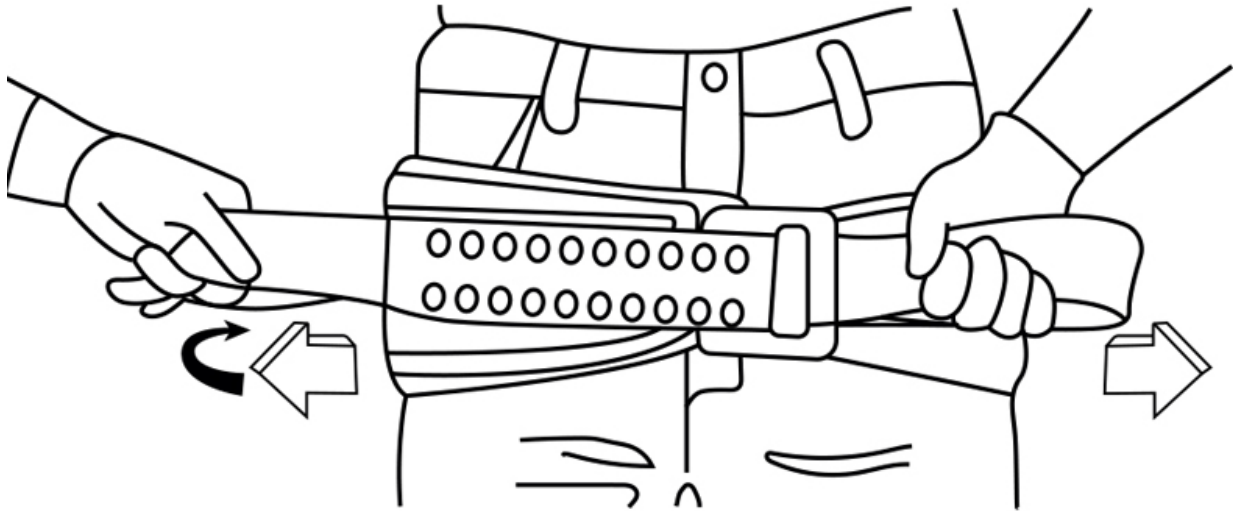

How to use the pelvic sling

The pelvic sling can be applied in 3 easy steps that requires no trimming, no cutting, no guessing

Step 1: Remove objects from the patients pocket or pelvic area. Place the pelvic sling black side up beneath the patient at the level of trochanters (hips).

Step 2: Place Blackstrap through the buckle and pull completely through.

Step3: Hold the Orange strap and pull the black strap in the opposite direction until you hear and feel the buckle click. Maintain tension immediately and press black strap onto the surface of the pelvic sling to secure. You may hear a second click as the sling secures.